⚠ INSTRUCTIONS FOR USE (Canada) – This leaflet contains important product use and safety information. Please read carefully and retain for future reference. Contact lenses are made of different materials and dimensions and should always be fitted by an eye care professional. It is essential that contact lens wearers see their eye care professional regularly, and follow their directions and all labeling instructions for proper use of contact lenses and lens care products.

PRODUCT NAME(s)

Focus® DAILIES®
 Focus® DAILIES® PROGRESSIVES
 Focus® DAILIES® Toric
 DAILIES® AquaComfort Plus™

DESCRIPTION

The lenses are available in various lens designs that are used for different types of vision correction.

- Focus® DAILIES® and DAILIES® AquaComfort Plus™ (neiflicon A) ONE-DAY CONTACT LENSES are available in a spherical lens design.
- Focus® DAILIES® PROGRESSIVES (neiflicon A) ONE-DAY CONTACT LENSES are available in a multifocal lens design.
- Focus® DAILIES® Toric (neiflicon A) ONE-DAY CONTACT LENSES are available in a toric design.

Each of the lens designs are available in a range of dimensions and prescriptive powers. The eye care professional will prescribe the appropriate lens design and prescriptive power (Rx) that's needed to correct vision in your right and left eye.

When placed on the cornea, the lenses act to correct vision by refocusing light rays on the retina.

The lens material is 69% water and 31% neiflicon A polymer (polyvinyl alcohol partially acetylated with N-torrnylmethyl acrylamide). For VISITINT® lenses, the colour additive copper phthalocyanine is added to the lens material to create a light blue edge to edge colour to make them easier to see when handling. Print marks on Focus® DAILIES® Toric lenses contain the colour additive phthalocyanine green.

Lenses are packaged in strips of five foil sealed blister packs containing isotonic phosphate-acetate buffered saline solution and are steam sterilized. These blister pack containers are attached to form a single strip. The package storage saline may contain up to 0.05% Poloxamer. In addition, the package storage saline for DAILIES® AquaComfort Plus™ lenses contains polyethylene glycol (PEG) and hydroxypropyl methylcellulose (HPMC). The base curve, lens power, lot number and expiration date are marked on the foil seal of each individual container. The diameter is marked on the rightmost container of each blister pack strip.

- Focus® DAILIES® lens models are available in: 30 packs - model # FD30, 90 packs - model # FD90, and 5 pack trials - model # FD5.
- Focus® DAILIES® PROGRESSIVES lens models are available in: 30 packs - model # FDP30 and 5 pack trials - model # FDP5.
- Focus® DAILIES® Toric lens models are available in: 30 packs - model # FDT30, 90 packs - model # FDT90, and 5 pack trials - model # FDT5.
- DAILIES® AquaComfort Plus™ lens models are available in: 30 packs - model # ACP30, 90 packs - model # ACP90, and 5 pack trials - model # ACP5.

Hereafter, Focus® DAILIES®, Focus® DAILIES® Progressives, Focus® DAILIES® Toric, and DAILIES® Aqua Comfort Plus™ will be referred to as DAILIES® unless product distinction is necessary.

INTENDED USE:

Use the lenses to improve the vision of non-diseased eyes. Spherical, toric, and multifocal lenses are available for persons who are nearsighted or farsighted, have astigmatism (irregular, oval shaped cornea) or presbyopia (age related loss of ability to focus on near tasks, such as reading).

All DAILIES® ONE-DAY CONTACT LENSES are for single use, daily disposable wear only.

RECOMMENDED WEAR AND REPLACEMENT SCHEDULE

Lenses are for single use, daily disposable wear only. Wearers should start each day with a fresh pair of lenses that are discarded at the end of that daily wearing period. Normal daily wear of lenses assumes a minimum of 6 hours of non-lens wear per 24-hour period. Optimum individual wearing schedule will vary.

In the interest of maintaining eye health, the wearing schedule should be determined by the eye care professional.

WHEN NOT TO WEAR LENSES

DO NOT USE DAILIES® ONE-DAY CONTACT LENSES if any of the following conditions exist:

- Allergy, inflammation, infection, redness or irritation on or around the eye or eyelids.
- The use of some medications, including eye medications. Always consult your eye care professional before using any medicines in your eyes.
- Systemic diseases that may be affected by or impact lens wear.
- Have certain types of allergic conditions.
- Excessively dry or dusty environments that make contact lens wear uncomfortable.
- Water sports without use of goggles.
- Inadequate tear film (dry eyes).
- If eyes become red or irritated.

Consult with the eye care professional specifically regarding these or other conditions.

IMPORTANT THINGS FOR WEARERS TO REMEMBER

- DO NOT WEAR DAILIES® ONE-DAY CONTACT LENSES while sleeping.
- Overnight wear of contact lenses has been shown to increase the risk of serious ocular complications and smoking increases these risks.
- It is essential that contact lens wearers see their eye care professional regularly. It is recommended contact lens wearers see their eye care professional once each year, or more frequently, if directed.
- Do not use lenses past the expiration date.
- Always carry spare lenses with you.
- Keep a note of the correct lens power for each eye. Before insertion, check that the lens power on each foil pack is correct for that eye.
- Cosmetics, lotions, soaps, creams or deodorants can all cause irritation if they come in contact with your lenses and should be used carefully.
- Remove lenses when exposed to harmful or irritating sprays, vapors and fumes.
- Consult your eye care professional about wearing lenses during sporting and water related activities. DO NOT USE saliva, tap water, distilled water, or homemade saline solutions to care for or when handling your lenses.
 - The use of tap and distilled water and/or exposure to nonsterile water while wearing contact lenses during water activities such as swimming, water skiing and hot tubs has been associated with Acanthamoeba keratitis, a corneal infection that is resistant to treatment and cure, and may increase the risk of:
 - Damage to the lenses by chemicals in the water
 - Loss of lenses
- Inform an employer that you wear contact lenses, especially if your job involves using eye protection equipment.

POSSIBLE PROBLEMS

While wearing contact lenses the eyes should look well, feel comfortable and vision should be clear.

Although contact lenses provide many benefits to the wearer, it is possible that problems can occur and may be first noticed as one or more of the following conditions:

- Feeling of something in the eye
- Uncomfortable lens
- Eye redness
- Sensitivity to light
- Burning, stinging, itching or watery eyes
- Reduced sharpness of vision
- Rainbows or halos around lights
- Increased eye secretions
- Severe or persistent dry eyes

WHAT TO DO IF A PROBLEM OCCURS

If any of the above symptoms occur **IMMEDIATELY REMOVE THE LENS(ES)**

- If the discomfort or problem stops, insert a fresh new lens.
- If the lens is in any way damaged, DO NOT put the lens(es) back on the eye. Replace with a new lens or consult the eye care professional.
- If the discomfort or problem persists after removal or after inserting a new lens, IMMEDIATELY and promptly remove lenses and consult the eye care professional.

A serious condition such as infection, corneal ulcer (ulcerative keratitis), corneal neovascularization, or iritis may be present. **These conditions could progress rapidly and, if left untreated, may lead to permanent loss of vision.** Less serious reactions such as abrasions, epithelial staining and bacterial conjunctivitis must be managed and treated properly to avoid complications. Professional identification of the problem and prompt treatment are necessary to avoid serious ocular complications.

SEASONAL ALLERGY WEARERS

Consult with the eye care professional regarding use of contact lenses and ocular allergies. A one-month subjective trial of Contact Lens Wearers with a history of seasonal allergic conjunctivitis was conducted during a month of expected high pollen count in various US cities. Information was collected about allergy related symptoms, wear time and comfort during lens wear.

Study results found that these contact lens wearers experienced fewer days of burning and redness when wearing Focus® DAILIES® as compared to a new pair of their usual lenses. The effects of allergy medications that may have been used during the study were not assessed.

ALL DAY COMFORT

A one month study of 188 subjects was conducted for the purpose of evaluating comfort and wearing time for Focus® DAILIES® soft contact lenses. End of day comfort was measured using a 0 to 10 scale where 0 was unacceptable and 10 was excellent. Wearing time was also recorded in hours of wear per day.

Baseline values for end of day comfort and average wearing time with the subject's pre-study lenses were 6.9 out of 10 and 13.5 hours, respectively. Study results found that the average end of day comfort for Focus® DAILIES® were statistically different compared to the baseline values collected from the pre-study lenses. As in this study, individual results may vary.

Reference: Bauman, E. (1997). Daily Disposables Versus Other Soft Lens Modalities. Optician 214: 33-35, 37.

HANDLING LENSES

The eye care professional should provide wearers with detailed contact lens wear, lens care, insertion and removal instructions.

- Always wash and dry hands before handling. Ensure the lens is right side out and that the correct lens for each eye is available. Inspect lenses prior to insertion. Do not insert lenses if damaged.
- Shake the blister pack gently prior to opening. Remove the lens from the blister pack by carefully pouring the lens onto the palm of your clean hand. Never use tweezers, sharp objects or fingernails to move, insert or remove lenses.
- To insert lenses:
 - Place a lens on the tip of your clean and dry right or left index finger and place the middle finger of the same hand close to your lower eyelashes and pull down the lower eyelid.
 - Use the fingers of the other hand to lift the upper right eyelid.
 - Place the lens directly on the eye (cornea) and gently roll your finger away from the lens.
 - Look down and slowly remove your right hand, releasing the lower lid.
 - Look straight ahead and slowly remove your left hand, releasing the upper lid.
 - Blink gently.
- If a lens de-centers on the eye, close eye and gently massage the eyelid to return the lens to the central position. If the problem persists, consult the eye care professional.
- When removing lenses, make sure hands are completely dry. Blink fully several times, then while looking up, slide the lens down onto the white part of the eye. Remove the lens by gently pinching between the thumb and forefinger.
- Never use tweezers, sharp objects or fingernails to move, insert or remove lenses.

EMERGENCY LENS CARE

Cleaning and disinfection of the lens for reuse is not recommended. Wearers should be reminded to have replacement lenses or back-up spectacles available at all times.

CARE FOR A DEHYDRATED LENS

If a DAILIES® ONE-DAY CONTACT LENS is exposed to air while off the eye it may become dry, brittle and permanently damaged. If this should occur, the lens should be discarded and replaced with a new one to avoid possible irritation or injury to the eye.

CARE FOR A STICKING OR TORN LENS

If a lens sticks (stops moving) or cannot be removed from the eye, apply 1 to 2 drops of a recommended lubricating or rewetting solution in accordance with package labelling. CIBA VISION recommends AQUI® Long-Lasting Comfort Drops. Blink forcefully several times, then while looking up slide the lens down onto the white part of the eye and remove the lens by pinching it between the thumb and forefinger. If the lens continues to stick, immediately consult the eye care professional.

If a lens tears in the eye it will feel uncomfortable. It is not possible to lose a contact lens or part of a contact lens behind the eye. Remove the pieces carefully by gently pinching them as you would do for normal lens removal. If the lens pieces do not seem to remove easily, do not pinch the eye tissue excessively. Rinse the eye thoroughly with sterile saline solution. If this does not help, contact your eye care professional for assistance. Your eye care professional can easily find and remove the lens.

GENERAL EMERGENCIES

If chemicals of any kind (household products, gardening solutions, laboratory chemicals, etc.) are splashed into the eyes:

- FLUSH EYES IMMEDIATELY WITH FRESH SALINE SOLUTION OR TAP WATER
 - REMOVE AND DISCARD THE LENS, AND IMMEDIATELY CONTACT THE EYE CARE PROFESSIONAL OR VISIT A HOSPITAL EMERGENCY ROOM WITHOUT DELAY.
- Additional information regarding emergency treatment may be provided on the product container label.

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©2009 CIBA VISION
CIBA VISION Corporation
CIBA VISION Canada, Inc.