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CIBA VISION Canada, Inc. 2150 Quayway Mesa, Mississauga, Ontario L5N 2M6 Canada

INSTRUCTIONS FOR USE (Canada) – This leaflet contains important product use and safety information. Please read carefully and retain for future reference. Contact lenses are made of different materials and dimensions and should always be fitted by an eye care professional. It is essential that contact lens wearers see their eye care professional regularly, and follow their directions and all labelling instructions for proper use of contact lenses and lens care products.

Rx only

PRODUCT NAME(S)

DURASOFT 3 LIFE TINT
 DURASOFT 3 COLORS
 DURASOFT 3 COLORS COMPLEMENTS
 DURASOFT 3 COLORBLENDS
 WILDEYES
 DURASOFT 3 OPTIFIT TORIC HANDLING TINT
 DURASOFT 3 OPTIFIT TORIC COLORS
 DURASOFT 3 OPTIFIT TORIC COLORS COMPLEMENTS
 DURASOFT 3 OPTIFIT TORIC COLORBLENDS
 DURASOFT 3 UV OPTIFIT TORIC HANDLING TINT

DESCRIPTION

DURASOFT 3 lenses are available in a spherical lens design. DURASOFT 3 OPTIFIT TORIC lenses are available in a toric lens design. Each of the lens designs are available in a range of dimensions and refractive powers. The eye care professional will prescribe the appropriate lens design that is needed to correct vision in your right and left eye.

When placed on the cornea, the lenses act to correct vision by refocusing light rays on the retina. COLORS, COLORS COMPLEMENTS, COLORBLENDS, and WILDEYES lenses also act to enhance or alter the apparent colour of the eye. These lenses are available with or without refractive power.

The lens material is 55% water and 45% pHEMA, a hydrophilic copolymer of 2-hydroxyethyl methacrylate and 2-ethoxyethyl methacrylate.

LIFE TINT / HANDLING TINT lenses contain the pigment phthalocyanine green to make them easier to see when handling. This tint does not affect iris colour.

COLORS, COLORS COMPLEMENTS, COLORBLENDS, and WILDEYES lenses are printed with inks containing the following colour additives (alone or in combination): iron oxides, chromium oxide greens, titanium dioxide [phthalocyaninato (2-)] copper, phthalocyanine green and carbazole violet. The pigments are not retained by lens handling and cleaning/disinfecting procedures. The optical and performance characteristics are not altered by the lens colouring process.

DURASOFT 3 UV OPTIFIT TORIC HANDLING TINT lenses contain a proprietary UV-absorbing monomer (CV BZT) which absorbs ultraviolet (UV) light.

WARNING: UV-absorbing contact lenses are NOT substitutes for protective UV-absorbing eyewear such as UV-absorbing goggles or sunglasses, because they do not completely cover the eye and surrounding area. You should continue to use UV-absorbing eyewear as directed.

NOTE: Long term exposure to UV radiation is one of the risk factors associated with cataracts. Exposure is based on a number of factors such as environmental conditions (altitude, geography, cloud cover) and personal factors (extent and nature of outdoor activities). UV-absorbing contact lenses help provide protection against harmful UV radiation. However, clinical studies have not been done to demonstrate that wearing UV-absorbing contact lenses reduce the risk of developing cataracts or other eye disorders. Consult your eye-care professional for more information.

All DURASOFT 3 lenses are packaged in individual, tamper evident sealed glass vials containing isotonic, sodium bicarbonate buffered saline and are steam sterilized. The glass vial is marked with the base curve, diameter, diopter, colour, cylinder and axis (if applicable), manufacturing lot number and expiration date. Do not use if the tamper evident seal has been broken.

Note: THE PACKAGING OF THIS PRODUCT CONTAINS DRY, NATURAL RUBBER.

The table below provides a listing of Health Canada Device Identifier Codes.

Product Name	Device Identifier	Product Name	Device Identifier
DURASOFT 3 LIFE TINT	D2LE	DURASOFT 3 OPTIFIT TORIC HANDLING TINT	D3LT
DURASOFT 3 COLORS	D3OP	DURASOFT 3 OPTIFIT TORIC COLORS	D3OF
DURASOFT 3 COLORS COMPLEMENTS	D3CO	DURASOFT 3 OPTIFIT TORIC COLORS COMPLEMENTS	D3OC
DURASOFT 3 COLORBLENDS	D3BC	DURASOFT 3 OPTIFIT TORIC COLORBLENDS	D3BC
WILDEYES	D3WE	DURASOFT 3 UV OPTIFIT TORIC HANDLING TINT	D3 X 3

INTENDED USE

Use the lenses to improve the vision of non-diseased eyes. Spherical and toric lenses are available for persons who are nearsighted (myopia) or farsighted (hyperopia) and may have astigmatism (irregular, oval shaped cornea). COLORS, COLORS COMPLEMENTS, COLORBLENDS, and WILDEYES lenses also enhance or alter the apparent colour of the eye.

RECOMMENDED WEAR AND REPLACEMENT SCHEDULE

In the interest of maintaining the health and safety of your eyes, the wear and replacement schedule should be determined by your eye care professional.

DURASOFT 3 lenses are recommended for daily wear (less than 24-hours per day, while awake). Lenses should be removed each day prior to sleep, with cleaning and disinfection prior to reinsertion on the eye.

To start lens wear, the eye care professional may prescribe an hourly schedule of increasing wear time per day over a period of several days or weeks until the wearer has successfully adapted to contact lens wear.

Normal daily wear of lenses assumes a minimum of 6 hours of non lens wear per 24 hour period, however individual wearing schedule will vary. The eye care professional should determine how many hours per day lenses may be worn.

CIBA VISION recommends annual replacement of your DURASOFT 3 lenses. Based on this recommendation, it is up to your eye care professional to determine the actual wear and replacement schedule that is best for you.

WHEN NOT TO WEAR LENSES

DO NOT USE DURASOFT 3 (pHemfilcon A) contact lenses when any of the following exists:

- Allergy, inflammation, infection or irritation on or around the eye or eyelids.
- The use of some medications, including eye medications. Always consult your eye care professional before using any medicines in your eyes.
- Have a systemic disease that may be affected by or impact contact lens wear.
- Have certain types of allergic conditions.
- Inadequate tear film (dry eyes).
- Water sports without use of goggles.
- If eyes become red or irritated.

Consult with the eye care professional specifically regarding these or other conditions.

IMPORTANT THINGS FOR WEARERS TO REMEMBER

- DO NOT WEAR DURASOFT 3 (pHemfilcon A) lenses while sleeping unless your eye care professional has prescribed the lenses for extended wear (overnight wear). Overnight wear of contact lenses has been shown to increase the risk of serious ocular complications and smoking increases these risks.
- It is essential that contact lens wearers see their eye care professional regularly. It is recommended contact lens wearers see their eye care professional twice each year or as recommended by the eye care professional.
- Do not use lenses past the expiration date and only use fresh, unexpired lens care products.
- Cosmetics, lotions, soaps, creams or deodorants can all cause irritation if they come in contact with your lenses and should be used carefully.
- Remove lenses when exposed to harmful or irritating sprays, vapors and fumes.
- Consult your eye care professional about wearing lenses during sporting activities, including swimming.
- UV absorbing contact lenses are NOT substitutes for protective UV absorbing eyewear such as UV absorbing goggles or sunglasses, because they do not completely cover the eye and surrounding area. You should continue to use UV absorbing eyewear as directed.
- Inform an employer that you wear contact lenses, especially if your job involves using eye protection equipment.
- Cosmetically tinted/coloured contact lenses may cause reduced visibility in low light conditions.

POSSIBLE PROBLEMS

While wearing contact lenses the eyes should look well, feel comfortable and vision should be clear.

Although contact lenses provide many benefits to the wearer, it is possible that problems can occur and may be first noticed as one or more of the following conditions:

- Feeling of something in the eye
- Uncomfortable lens
- Eye redness
- Sensitivity to itching
- Burning, stinging, light or watery eyes
- Reduced sharpness of vision
- Rainbows or halos around lights
- Increased eye secretions
- Severe or persistent dry eyes

WHAT TO DO IF A PROBLEM OCCURS

If any of the above symptoms occur:

IMMEDIATELY REMOVE THE LENSES.

- If the discomfort or problem stops, then look closely at the lens(es):
 - If the lens(es) is in any way damaged, DO NOT put the lens(es) back on the eye. Replace with a new lens or consult the eye care professional.
 - If the lens(es) have dirt, an eye lash or other foreign body on it, or the problem stops at the lenses appear undamaged, thoroughly clean, rinse, and disinfect prior to reinsertion.
- If the discomfort or problem continues after removing lens(es) or upon reinsertion, IMMEDIATELY remove the lens(es) and contact the eye care professional for identification of the problem and prompt treatment to avoid serious eye damage.

A serious condition such as infection, corneal ulcer (ulcerative keratitis), corneal vascularization, or iritis may be present. These conditions could progress rapidly and may lead to permanent loss of vision. Less serious reactions such as abrasions, infiltrates, and bacterial conjunctivitis must be managed and treated carefully to avoid more serious complications. Professional identification of the problem and prompt treatment are necessary to avoid serious ocular complications.

HANDLING LENSES

The eye care professional should provide wearers with detailed contact lens wear, lens care, insertion and removal instructions.

Always wash and dry hands before handling. Ensure the lens is right side out and that the correct lens for each eye is available. Inspect lenses prior to insertion. Do not insert lenses if damaged.

- To insert lenses:
 - Place a lens on the tip of your clean and dry right or left index finger, place and keep the middle finger of the same hand close to your lower eyelashes and pull down the lower eyelid.
 - Use the fingers of the other hand to lift the upper eyelid.
 - Place the lens directly on the eye (cornea) and gently roll your finger away from the lens.
 - Look down and slowly remove your hand, releasing the lower lid.
 - Look straight ahead and slowly remove your other hand, releasing the upper lid.
 - Blink gently.
- To remove lenses:
 - Make sure hands are completely dry.
 - Blink fully several times.
 - While looking up, slide the lensing gently onto the white part of the eye.
 - Remove the lens by pinching gently between the thumb and forefinger.
- If a lens decenters on the eye, close eye and gently massage the eyelid to return the lens to the central position. If the problem persists, consult the eye care professional.
- Never use tweezers, sharp objects or fingernails to move, insert or remove lenses.

TAKING CARE OF LENSES

- Good hygiene and correct care of lenses are essential.
- DISINFECT lenses each time they are removed to destroy harmful germs and to ensure safe and comfortable contact lens wear. Use a chemical, not heat, disinfection system.
- A separate CLEANING & RINSING step may be recommended by the eye care professional to remove contaminants from the lens surface. In addition, lenses may also require periodic enzymatic cleaning to remove protein build-up.
- Remove one lens first (always the same lens first to avoid mix-ups) and place the lens into the correct chamber of the lens storage case. Repeat this procedure for the second lens.
- The eye care professional should recommend an appropriate lens care system. CIBA VISION recommends the following for use with DURASOFT 3 lenses:
 - Hydrogen peroxide disinfecting system – Clear Care
 - Chemical cleaning & disinfecting solution – SOLOCARE AQUA® All-In-One Solution
 - Cleaner – AOFLOW® Extra Strength Daily Cleaner
 - Protein Removal – Unizyme® Enzymatic Cleaner
 - Lubricating & rewetting drops – Aquify® Long-Lasting Comfort Drops.
- To store lenses, clean, disinfect, neutralize, rinse, and leave them in the closed/unopened lens storage case until ready to wear. If the lenses have been stored in the unopened lens storage case for more than 24 hours, repeat the cleaning, disinfecting, neutralizing and rinsing procedures using fresh solutions prior to wearing the lenses.
- Contact lens cases require proper use, cleaning and replacement at regular intervals. To prevent contamination and avoid serious eye injury, follow the manufacturer's instructions for use and care of the lens case and replace the lens case according to the manufacturer's instructions, or as recommended by the eye care professional.

Important notes:

- DO NOT USE saline, tap water, distilled water, or homemade saline solutions to care for or when handling your lenses. The use of tap and distilled water has been associated with Acanthamoeba keratitis, a corneal infection that is resistant to treatment and cure.
- DO NOT USE products designed only for hard or rigid gas permeable lenses.
- NEVER USE HEAT with lens care solutions or to disinfect lenses.
- Refer to the package inserts of each individual lens care product for important safety information and complete directions.

CARE FOR A DEHYDRATED LENS

A lens must move freely on the eye for the continued health of the eye. If the lens sticks to the eye, do not rub the eye, apply 2-3 drops of a sterile saline solution recommended for use in eyes. Wait until the lens begins to move freely on the eye before removing it. If non-movement of the lens continues after several minutes, immediately consult the eye care professional.

CARE FOR A STICKING LENS

If the lens sticks (stops moving) or begins to dry on the eye, the patient should be instructed to apply several drops of the recommended lubricating or rewetting solution directly to the eye and wait until the lens begins to move freely on the eye before removing it. If non-movement of the lens continues after several minutes, the patient should be instructed to immediately consult the eye care professional.

GENERAL EMERGENCIES

If chemicals of any kind (household products, gardening solutions, laboratory chemicals) are splashed into the eyes:

- FLUSH EYES IMMEDIATELY WITH FRESH SALINE SOLUTION OR TAP WATER.
- REMOVE AND DISCARD THE LENS, AND IMMEDIATELY CONTACT YOUR EYE CARE PROFESSIONAL OR VISIT A HOSPITAL EMERGENCY ROOM WITHOUT DELAY.