**INSTRUCTIONS FOR USE (Canada)** – This leaflet contains important product use and safety information. Please read carefully and retain for future reference. Contact lenses are made of different materials and dimensions and should always be fitted by an eye care professional. It is essential that contact lens wearers see their eye care professional regularly, and follow their directions and all labeling instructions for proper use of contact lenses and lens care products.

**PRODUCT NAME(s)**  
FreshLook® ONE-DAY (nelfilcon A) Color Soft Contact Lenses

**DESCRIPTION**  
The lenses are available in a spherical lens design in a range of dimensions and prescriptive powers. The eye care professional will prescribe the appropriate lens design that's needed to correct vision in your right and left eye.

When placed on the cornea contact lenses act to correct vision by refocusing light rays on the retina.

The lens material is approximately 69% water and 31% nelfilcon A polymer (polyvinyl alcohol partially acetalized with N-formylmethyl acrylamide). FreshLook ONE-DAY (nelfilcon A) Color Soft Contact Lenses are printed with an intermittent coating containing a combination of the following pigments: iron oxides, titanium dioxide, [phtalocyaninato(2-)] copper, chromium oxide and carbazole violet.

Lenses are packaged in strips of five foil sealed blister packs containing isotonic phosphate-acetate buffered saline solution and are steam sterilized. The package storage saline may also contain up to 0.02% Poloxamer 108. The base curve, lens power, lot number and expiration date are marked on the foil seal of each individual container. The diameter is marked on the rightmost container of each strip of five blister packs.

**INTENDED USE**  
Use FreshLook® ONE-DAY (nelfilcon A) Color Soft Contact Lenses to improve the vision of nearsighted (myopia), and farsighted (hyperopia) persons with non-diseased eyes. The FreshLook® ONE-DAY (nelfilcon A) Color Soft Contact Lenses also act to enhance or alter the apparent color of the eye.

The lenses may be prescribed for daily disposable wear with removal for replacement with a fresh new lens.

**RECOMMENDED WEAR AND REPLACEMENT SCHEDULE**  
In the interest of maintaining the health and safety of your eyes, the wear and replacement schedule should be determined by your eye care professional.

- Lenses are for single use, daily disposable wear only. Wearers should start each day with a fresh pair of lenses that are discarded at the end of the daily wearing period.
- To start lens wear, the eye care professional may prescribe an hourly schedule of increasing wear time per day over a period of several days or weeks until the wearer has successfully adapted to contact lens wear.
- Normal daily wear of lenses assumes a minimum of 6 hours of non lens wear per 24 hour period, however individual wearing schedule will vary. The eye care professional should determine how many hours per day lenses may be worn.

**WHEN NOT TO WEAR LENSES**

**DO NOT USE FreshLook® ONE-DAY (nelfilcon A) Color Soft Contact Lenses when any of the following exists:**

- Inflammation or infection in or around the eye or eyelids.
- Excessive dryness of the eyes that makes contact lens wear uncomfortable.
- Reduced corneal sensitivity.
- Systemic diseases that may be affected by or impact lens wear.
- Allergic conditions, reactions or ocular irritation caused or exaggerated by lens wear or certain preservatives in lens care products, such as rewetting drops.
- The use of any medication, including some eye medications, that shouldn't be used with, or interferes with contact lens wear.
- If eyes become red or irritated.

Consult with the eye care professional specifically regarding these or other conditions.

**IMPORTANT THINGS FOR WEARERS TO REMEMBER**

- DO NOT WEAR FreshLook® ONE-DAY (nelfilcon A) Color Soft Contact Lenses while sleeping. Overnight wear of contact lenses has been shown to increase the risk of serious ocular complications and smoking increases these risks.
- It is essential that contact lens wearers see their eye care professional regularly. It is recommended contact lens wearers see their eye care professional once each year, or as recommended by the eye care professional.
- Do not use lenses past the expiration date.
- Always carry spare lenses with you.
- Keep a note of the correct lens power for each eye. Before insertion, check that the lens power on each foil pack is correct for that eye.
- Cosmetics, lotions, soaps, creams or deodorants can all cause irritation if they come in contact with your lenses and should be used carefully.
- Remove lenses when exposed to harmful or irritating sprays, vapors and fumes.
- Consult your eye care professional about wearing lenses during sporting activities, including swimming. Exposure to non sterile water such as tap water or while wearing contact lenses during activities such as swimming, water skiing and hot tubs may increase the risk of:
  - Eye infection, including but not limited to Acanthamoeba keratitis
  - Damage to the lenses by chemicals in the water
  - Loss of lenses
- Inform an employer that you wear contact lenses, especially if your job involves using eye protection equipment.

**POSSIBLE PROBLEMS**

While wearing contact lenses the eyes should look well, feel comfortable and vision should be clear.

Although contact lenses provide many benefits to the wearer, it is possible that problems can occur and may be first noticed as one or more of the following conditions:

- Feeling of something in the eye
- Uncomfortable lens
- Eye redness

- Sensitivity to light
- Burning, stinging or itching or watery eyes
- Reduced sharpness of vision
- Rainbows or halos around lights
- Increased eye secretions
- Severe or persistent dry eyes

**WHAT TO DO IF A PROBLEM OCCURS**  
If any of the above symptoms occur IMMEDIATELY REMOVE THE LENSES.

- If the discomfort or problem stops, insert a fresh new lens.
- If the discomfort or problem persists after removal or after inserting a new lens, IMMEDIATELY and promptly remove the lens(es) and contact the eye care professional.

A serious condition such as infection, corneal ulcer (ulcerative keratitis), corneal vascularization, or iritis may be present. These conditions could progress rapidly and may lead to permanent loss of vision. Less serious reactions such as abrasions, infiltrates, and bacterial conjunctivitis must be managed and treated carefully to avoid more serious complications. Professional identification of the problem and prompt treatment are necessary to avoid serious ocular complications.

**SEASONAL OCULAR ALLERGY WEARERS**  
Consult with the eye care professional regarding use of contact lenses and ocular allergies. A one-month subjective trial of contact lens wearers with a history of seasonal allergic conjunctivitis was conducted during a month of expected high pollen count in various US cities. Information was collected about allergy related symptoms, wear time and comfort during lens wear.

Study results found that these contact lens wearers experienced fewer days of burning and redness when wearing (nelfilcon A) contact lenses as compared to a new pair of their usual lenses. The effects of allergy medications that may have been used during the study were not assessed.

**HANDLING LENSES**

The eye care professional should provide wearers with detailed contact lens wear, lens care, insertion and removal instructions.

- Always wash and dry hands before handling. Ensure the lens is right side out and that the correct lens for each eye is available. Inspect lenses prior to insertion. Do not insert lenses if damaged.
- To insert lenses:
  - Place a lens on the tip of your clean and dry right or left index finger. Place the middle finger of the same hand close to your lower eyelashes and pull down the lower eyelid.
  - Use the fingers of the other hand to lift the upper eyelid.
  - Place the lens directly on the eye (cornea) and gently roll your finger away from the lens.
  - Look down and slowly remove your hand, releasing the lower lid.
  - Look straight ahead and slowly remove your other hand, releasing the upper lid.
  - Blink gently.
- To remove lenses:
  - Make sure hands are completely dry.
  - Blink fully several times.
  - While looking up, slide the lens down onto the white part of the eye.
  - Remove the lens by pinching gently between the thumb and forefinger.
- If the lens tears in your eye:
  - It will feel uncomfortable. Do not panic as it is not possible to lose a contact lens or part of a contact lens behind the eye. Remove the pieces carefully by gently pinching them as you would do for normal lens removal. If the lens pieces do not seem to remove easily, do not pinch the eye tissue excessively. Rinse the eye thoroughly with sterile saline solution. If this does not help, contact your eye care professional for assistance. Your eye care professional can easily find and remove the lens.
  - If a lens decenters on the eye, close eye and gently massage the eyelid to return the lens to the central position. If the problem persists, consult the eye care professional.
  - Never use tweezers, sharp objects or fingernails to move, insert or remove lenses.

**EMERGENCY LENS CARE**

Cleaning and disinfection of the lens for re-use is not recommended. Wearers should be reminded to have replacement lenses or back-up spectacles available at all times.

**CARE FOR A STICKING LENS**

A lens must move freely on the eye for the continued health of the eye. If a lens sticks (stops moving) or begins to dry on the eye, apply a lubricating solution recommended for soft lenses according to the manufacturer's instructions for use. Wait until the lens begins to move on the eye before removing it. If non-movement of the lens continues, immediately consult the eye care professional.

**GENERAL EMERGENCIES**

- If chemicals of any kind (household products, gardening solutions, laboratory chemicals, etc.) are splashed into the eyes:
  - FLUSH EYES IMMEDIATELY WITH TAP WATER OR FRESH SALINE SOLUTION.
  - REMOVE AND DISCARD THE LENS, AND IMMEDIATELY CONTACT AN EYE CARE PROFESSIONAL OR VISIT A HOSPITAL EMERGENCY ROOM WITHOUT DELAY.

**Additional Information**

- Cosmetically tinted contact lenses may cause reduced visibility in low light conditions.

The table below provides a listing of Health Canada Device Identifier Codes.

Product Name	Additional Product Description	Device Identifier
FreshLook ONE-DAY	10 pack	FLOD10
FreshLook ONE-DAY	15 pack	FLOD15
FreshLook ONE-DAY	30 pack	FLOD30

CIBA Vision Corporation  
11460 Johns Creek Parkway  
Duluth, Georgia USA 30097  
www.cibavision.com

CIBA Vision Canada, Inc  
2150 Torquay Mews,  
Mississauga, Ontario  
L5N 2M6 Canada

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