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**INSTRUCTIONS FOR USE** – This leaflet contains important product use and safety information. Please read carefully and retain for future reference.

Contact lenses are made of different materials and dimensions and should always be fitted by an eye care professional. It is essential that contact lens wearers see their eye care professional regularly, and follow their directions and all labeling instructions for proper use of contact lenses and lens care products.

**PRODUCT NAME(s)**  
AIR OPTIX™ NIGHT & DAY® AQUA

**DESCRIPTION**  
AIR OPTIX NIGHT & DAY AQUA (Ioftrifilcon A) soft contact lenses are available in a spherical design in a range of dimensions and prescriptive powers [measured in diopters (D)]. The eye care professional will prescribe the appropriate prescriptive power (Rx) that's needed to correct vision in your right and left eye.

When placed on the cornea (front part of the eye), the lenses act to correct vision by refocusing light rays on the retina (back part of the eye). For therapeutic use, the lenses act as a bandage to protect the cornea.

The lens material is approximately 24% water and 76% Ioftrifilcon A, a fluoro-silicone containing hydrogel that is surface treated to wet with your tears. Lenses contain the color additive, a light blue handling tint, which makes them easier to see when handling. Each lens is packaged in a foil-sealed blister pack containing isotonic phosphate buffered saline with 1% Copolymer 845 and is steam sterilized. The package is marked with the base curve, diameter, diopter power, manufacturing lot number and expiration date. Lenses are available in an 8.4mm base curve (Model/Device ID CIBA Z4u) and an 8.6mm base curve (Model/Device ID CIBA Z6u).

Use only if pack and printed foil are intact.

**INTENDED USE:**  
*Vision Correction*

AIR OPTIX NIGHT & DAY AQUA (Ioftrifilcon A) soft contact lenses are used to improve vision in persons who are nearsighted (myopia) or farsighted (hyperopia). The lenses may be worn for daily wear or extended wear for up to 30 nights and days of continuous wear as recommended by your eye care professional.

#### *Therapeutic Use*

In addition, Ioftrifilcon A lenses are for therapeutic use. They may be used as a bandage to protect the cornea, to relieve corneal pain, to aid corneal healing in the treatment of various acute or chronic ocular conditions, and for post-surgical conditions resulting from cataract extraction and corneal surgery. Ioftrifilcon A soft contact lenses for therapeutic use can also provide optical correction during healing if required.

#### **RECOMMENDED WEAR AND REPLACEMENT SCHEDULE**

In the interest of maintaining the health and safety of your eyes, the wear and replacement schedule should be determined by your eye care professional.

- AIR OPTIX NIGHT & DAY AQUA lenses may be prescribed for daily wear or extended wear for up to 30 nights of continuous wear, with removal for disposal, or cleaning and disinfection prior to reinsertion, as recommended by your eye care professional. However, not everyone can reach the maximum wear period. Clinical studies have shown that approximately 90% of patients dispensed lenses are able to achieve some period of extended wear and approximately 65% are able to wear their lenses for the full 30 nights extended wear.
- Your eye care professional may recommend daily wear, or a shorter period of continuous wear depending on your individual needs, and you should always adhere to his or her recommendations. If you need to wear lenses for less than 30 nights your eye care professional will advise you on the appropriate use of contact lens care products, or may recommend disposable use.
- For extended wear, lenses can be worn continuously for up to 30 nights. Once lenses are removed, your eyes should have a rest without lens wear for at least one overnight. Your eye care professional will tell you how long to rest your eyes in between wearing periods and will also recommend a replacement period and appropriate lens care products that are right for you.
- For daily wear (less than 24 hours per day, while awake), normal daily wear assumes a minimum of 6 hours of non lens wear per 24 hour period, however individual wearing schedule will vary. The eye care professional should determine how many hours per day lenses may be worn.
- For therapeutic use, close professional supervision is necessary. The eye care professional should provide specific instructions regarding lens wear, removal and insertion.
- CIBA VISION recommends monthly replacement for Ioftrifilcon A lenses. Based on this recommendation, it is up to your eye care professional to determine the actual wear and replacement schedule that is best for you.

#### **WHEN NOT TO WEAR LENSES**

**DO NOT USE AIR OPTIX NIGHT & DAY AQUA (Ioftrifilcon A) contact lenses when any of the following exist:**

- Allergy, inflammation, infection or irritation on or around the eye or eyelids
- Periods of poor health, such as during severe colds or influenza
- The use of some medications, including eye medications. Always consult your eye care professional before using any medicines in your eyes.
- A systemic disease that may be affected by or impact contact lens wear
- Inadequate tear film (dry eyes)
- Excessively dry or dusty environments that make contact lens wear uncomfortable
- Water sports without use of goggles
- If eyes become red or irritated

Consult with your eye care professional specifically regarding these or other conditions.

For THERAPEUTIC USE, the eye care professional may prescribe Ioftrifilcon A lenses to aid in the healing process of certain corneal conditions.

#### **POSSIBLE PROBLEMS**

While wearing lenses your eyes should look well, feel comfortable, and your vision should be clear.

Although contact lenses provide many benefits to the wearer, it is possible that problems can occur. Check your eyes every day for the following conditions:

- Moderate to severe eye pain not relieved by removing the lens
- Feeling of something in the eye
- Uncomfortable lens
- Eye redness
- Sensitivity to light
- Burning, stinging, itching or watering eyes
- Reduced sharpness of vision
- Rainbows or halos around lights
- Unusual eye secretions
- Severe or persistent dry eyes
- Feeling of dryness

These symptoms, if ignored, can lead to more serious complications.

During therapeutic use, an adverse effect may be due to the original disease or injury or may be due to the effects of wearing a contact lens. Your eye care professional should discuss the possibility that the existing disease or condition might become worse when a soft contact lens for therapeutic use is used to treat an already diseased or damaged eye. The patient should be instructed to avoid serious eye damage by contacting the eye care practitioner IMMEDIATELY if there is any increase in symptoms while wearing the lens.

#### **WHAT TO DO IF A PROBLEM OCCURS**

When lenses are being used for vision correction in healthy eyes, if any of the above signs or symptoms occur:

- IMMEDIATELY REMOVE THE LENS.
- If the discomfort or problem stops, then look closely at the lens for problems.
- If lens is damaged, DO NOT put the lens back on the eye. Replace with a new lens or consult the eye care professional.
- If the lens has dirt, an eyelash or other spots on it, thoroughly clean, rinse and disinfect it before insertion. If the problem returns on reinsertion, the lens should be discarded and a new lens placed on the eye.
- If dryness persists, consult the eye care professional.
- If the condition does not stop after the lens has been removed, or returns after insertion of a new lens, IMMEDIATELY REMOVE THE LENS AND CONTACT THE EYE CARE PROFESSIONAL. Prompt attention is necessary or serious eye problems, including corneal ulcers, can develop. Loss of vision or scarring of the cornea may result.

During therapeutic use, if problems occur or worsen, YOU SHOULD IMMEDIATELY CONTACT YOUR EYE CARE PROFESSIONAL and follow the directions provided.

#### **ADDITIONAL INFORMATION**

- The risk of microbial keratitis, a rare but serious condition resulting in corneal scarring and leading to permanent reduction of vision, has been shown to be greater among users of extended wear contact lenses (0.2% or 20/10,000 wearers per year) than among users of daily wear contact lenses (0.04% or 4/10,000 wearers per year).
- Possible problems may be reduced by recognizing the signs and following the advice under What to do if a Problem Occurs.
- For therapeutic use, medications necessary for treatment should be used with caution under close supervision by the eye care professional.

#### **IF YOU SMOKE**

Smoking increases the risk of serious problems with contact lens wear, especially when lenses are worn overnight or while sleeping. If you smoke, be sure to inform your eye care professional.

#### **IMPORTANT THINGS TO REMEMBER**

- It is recommended that contact lens wearers see their eye care professional regularly as directed. CIBA VISION recommends contact lens wearers see their eye care professional at least twice per year, or as recommended by the eye care professional. More frequent visits will be required for therapeutic use and may be needed for first time contact lens wearers or starting continuous wear.
- You should always have a pair of spectacles (eyeglasses) that you are willing to wear if a problem occurs with your contact lenses.
- Never exceed the prescribed wearing schedule, even if lenses still feel comfortable.
- Do not use lenses after the expiration date.
- Know the correct lens power for each eye.
- Cosmetics, lotions, soaps, creams or deodorant can all cause irritation if they come in contact with your lenses. Use them carefully.
- Avoid spraying aerosols such as hair spray near your eyes while wearing your lenses as they may cause irritation.
- Consult your eye care professional about wearing lenses during sporting activities including swimming.
- Remove your lenses when exposed to nonxious or irritating vapors.
- Never expose contact lenses to any non-sterile water (including tap water) as microbial contamination can occur, which may lead to permanent eye damage.

In addition, for therapeutic use:

- Your vision may be compromised during contact lens therapy. This may require supplemental spectacles for vision correction, and limitations to daily activities, such as driving. You should discuss this with your eye care practitioner.
- Exposure to water may temporarily affect the fit of the lens and could lead to a serious eye infection. Do not get water in the eye while bathing or showering. While wearing contact lenses for therapeutic use, discontinue water activities such as swimming or use of hot tubs.

#### **HANDLING YOUR LENSES**

The eye care professional should provide wearers with detailed contact lens wear, care, insertion and removal instructions.

- Always wash and dry your hands before handling lenses.
- Peel back the foil lid and, gently pour the lens onto the palm of your clean hand. Ensure that the lens is right side out and that the correct lens is available for each eye.
- Inspect lenses prior to insertion. Do not insert if damaged.
- To insert lenses:
  - Place a lens on the tip of your clean and dry right or left index finger, place the middle finger of the same hand close to your lower eyelashes and pull down the lower eyelid.
  - Place the fingers of the other hand to lift the upper eyelid.
  - Place the lens directly on the eye (cornea) and gently roll your finger away from the lens.
  - Look down and slowly remove your hand, releasing the lower lid.
  - Look straight ahead and slowly remove your other hand, releasing the upper lid.
  - Blink gently.
- To remove lenses:
  - When removing lenses, make sure hands are completely dry.
  - Blink fully several times. While looking up, slide the lens down onto the white part of the eye.
  - Remove the lens by pinching gently between the thumb and forefinger.
  - If a lens de-centers on the eye, close eye and gently massage the eyelid to return the lens to the central position. If the problem persists, consult the eye care professional.
  - Never uses tweezers, sharp objects or fingernails to move, insert or remove lenses from the eye or the lens container.

For therapeutic use, your eye care professional may instruct you NOT to insert or remove your lenses by yourself. Under these circumstances, the eye care professional will insert and remove your lenses for you.

#### **TAKING CARE OF LENSES**

- Good hygiene and correct care of lenses are essential.
- DO NOT use saliva, tap water, distilled water or homemade saline solution for any purpose in caring for your lenses. The use of these solutions has been associated with serious eye infections, including Acanthamoeba keratitis, a corneal infection which is resistant to treatment and cure.
- DISINFECT lenses each time they are removed to destroy harmful germs and to ensure safe and comfortable contact lens wear. Use a chemical, not heat, disinfecting system.
- A separate CLEANING & RINSING step may be recommended by the eye care professional to remove contaminants from the lens surface.
- Remove one lens first (always the same lens first to avoid mix-ups) and place the lens into the correct chamber of the lens storage case. Repeat this procedure for the second lens.
- The eye care professional should recommend an appropriate lens care system. CIBA VISION recommends the following lens care products:
  - Hydrogen peroxide disinfecting systems - Clear Care®
  - Chemical cleaning & disinfecting solutions – SOLOCARE AQUA™ All-In-One Solution
  - Cleaners – ADFLOW™ Extra Strength Daily Cleaner
  - Lubricating & rewetting drops – AQUITY® Long-Lasting Comfort Drops
- To store lenses, clean, disinfect, neutralize, rinse, and leave them in the closed/unopened lens storage case until ready to wear. If the lenses have been stored in the unopened lens storage case for more than 24 hours, repeat the cleaning, disinfecting, neutralizing and rinsing procedures using fresh solutions prior to wearing the lenses.
- Contact lens cases require proper use, cleaning and replacement at regular intervals. To prevent contamination and avoid serious eye injury, follow the manufacturer's instructions for use and care of the lens case and replace the lens case according to the manufacturer's instructions, or as recommended by the eye care professional.
- If a lens is exposed to air while off the eye it may become dry and permanently damaged. If this should occur, discard the lens and use a new one to avoid possible irritation or injury to the eye.
- Do not use heat or products designed only for hard or rigid gas permeable lenses.
- Refer to the package inserts of each individual lens care product for important safety information and complete directions.

#### **CARE FOR A STICKING LENS**

If the lens sticks (stops moving) or begins to dry on the eye, apply several drops of a recommended lubricating solution (used in accordance with package labeling). Wait until the lens begins to move freely on the eye before attempting to remove it. If the lens continues to stick, IMMEDIATELY consult the eye care professional.

#### **EMERGENCIES**

If chemicals of any kind (household products, gardening solutions, laboratory chemicals, etc.) are splashed into the eyes:

- Flush eyes immediately with tap water or fresh saline solution.
- Remove and discard the lens, and immediately contact the eye care professional or a visit a hospital emergency room without delay.

CIBA VISION Corporation  
21460 Johns Creek Parkway  
Duluth, Georgia 30097 USA  
www.cibavision.com

CIBA VISION Canada, Inc  
11500 Torquay Mews,  
Mississauga, Ontario  
L5N 2M6 Canada

Print Date: Jan 2009

Printed in: Canada

D7450A/C001070