INSTRUCTIONS FOR USE – This leaflet contains important product use and safety information. Please read carefully and retain for future reference. Contact lenses are made of different materials and dimensions and should always be fitted by an eye care professional. It is essential that contact lens wearers see their eye care professional regularly, and follow their directions and all labeling instructions for proper use of contact lenses and lens care products.

For Only

PRODUCT NAME

SoftPerm®

DESCRIPTION

The SoftPerm (synergicon A) contact lens is a clear, spherical lens available in a range of dimensions and prescriptive powers. When placed on the cornea, the lenses act to correct vision by refocusing light rays on the retina.

The SoftPerm contact lens consists of a rigid circular shell (< 0.2% water) with a soft hydrophilic skirt (25% water). The lenses are packaged in individual, tamper evident sealed glass vials containing isotonic sodium bicarbonate buffered saline solution and are sterilized by gamma irradiation. The glass vial is marked with the base curve, dioptric power, diameter, manufacturing lot number and expiration date. Do not use if the tamper evident seal has been broken.

Note: THE PACKAGING OF THIS PRODUCT CONTAINS DRY, NATURAL RUBBER (the vial stopper).

INTENDED USE

SoftPerm contact lenses are indicated for **daily wear** for the correction of vision in persons with non-diseased eyes who are near sighted or far sighted with minimal or some amount of astigmatism or who require additional correction for special conditions, e.g. keratoconus, irregular astigmatism.

RECOMMENDED WEAR AND REPLACEMENT SCHEDULE

CIBA Vision recommends that SoftPerm lenses routinely be replaced every 12 to 18 months or more frequently, as determined by the eye care professional.

In the interest of maintaining eye health, the wearing schedule should be determined by the eye care professional.

- DAILY WEAR** - (less than 24 hours, while awake)
To start lens wear, the eye care professional may prescribe an hourly schedule of increasing wear time per day over a period of several days or weeks until the wearer has successfully adapted to contact lens wear.

Studies have not been conducted to show SoftPerm lenses are safe to wear during sleep, therefore these contact lenses should never be worn while sleeping. Normal daily wear of lenses assumes a minimum of 6 hours of non-lens wear per 24-hour period, however individual wearing schedule will vary. The eye care professional should determine how many hours per day lenses may be worn. Regular examinations at scheduled intervals, as determined by the eye care professional, are also extremely important.

WHEN NOT TO WEAR LENSES

DO NOT USE SoftPerm contact lenses if any of the following conditions exist:

- Allergy, inflammation, infection or irritation on or around the eye or eyelids.
- The use of some medications, including eye medications. Always consult your eye care professional before using any medicines in your eyes.
- Excessively dry or dusty environments that make contact lens wear uncomfortable.
- Water sports without use of goggles.
- Inadequate tear film (dry eyes).

Consult with the eye care professional specifically regarding these or other conditions.

IMPORTANT THINGS FOR WEARERS TO REMEMBER

- DO NOT WEAR** SoftPerm Contact Lenses **WHILE SLEEPING**. The risk of serious complications is greater for daily wear users who wear their lenses overnight, and smoking increases these risks.
- It is essential that contact lens wearers see their eye care professional regularly as directed.
- Do not use lenses past the expiration date and only use fresh, unexpired lens care products.
- Cosmetics, lotions, soaps, creams or deodorants can all cause irritation if they come in contact with your lenses and should be used carefully.
- Remove lenses when exposed to harmful or irritating sprays, vapors and fumes.
- Consult the eye care professional about wearing lenses during sporting activities, including swimming.
- Inform an employer that you wear contact lenses, especially if your job involves using eye protection equipment.
- SoftPerm contact lenses must never be immersed in any solution containing the disinfectant chlorhexidine as this may seriously interfere with lens wettability and comfort on the eyes.

POSSIBLE PROBLEMS

While wearing contact lenses the eyes should look well, feel comfortable and vision should be clear. Although contact lenses provide many benefits to the wearer, it is possible that problems can occur and may be first noticed as one or more of the following conditions:

- Feeling of something in the eye
- Uncomfortable lens
- Eye redness
- Sensitivity to light
- Burning, stinging, itching or watering eyes
- Reduced sharpness of vision
- Rainbows or halos around lights
- Increased eye secretions
- Severe or persistent dry eyes

WHAT TO DO IF A PROBLEM OCCURS

If any of the above symptoms occur:

- IMMEDIATELY REMOVE THE LENS(ES)**
- If the discomfort or problem stops, then look closely at the lens(es).
- If the lens appears damaged, **DO NOT** put the lens back on the eye. Replace with a new lens(es) or consult the eye care professional.
- If the lens(es) have dirt, an eyelash, or other foreign body on it, or the problem stops and the lenses appear undamaged, thoroughly clean, rinse and disinfect the lens(es); then reapply to the eye.
- If the discomfort or problem persists after removal or upon reinsertion, **IMMEDIATELY** and promptly remove lenses and consult the eye care professional.

A serious condition such as infection, corneal ulcer (ulcerative keratitis), corneal neovascularization, or iritis may be present. **These conditions could progress rapidly and may lead to permanent loss of vision.** Less serious reactions such as abrasions, epithelial staining and bacterial conjunctivitis must be managed and treated properly to avoid complications. Professional identification of the problem and prompt treatment are necessary to avoid serious ocular complications.

HANDLING LENSES

The eye care professional should provide wearers with detailed contact lens wear, lens care, insertion and removal instructions. Good hygiene and correct care of lenses and the lens case are essential for successful contact lens wear.

- Always wash and dry hands before handling. Ensure the lens is right side out and that the correct lens for each eye is available. Inspect lenses prior to insertion. Do not insert lenses if damaged.
- When removing lenses, make sure hands are completely dry. Blink fully several times, then while looking up, slide the lens down onto the white part of the eye. Remove the lens by pinching between the thumb and forefinger.
- If a lens de-centers on the eye, close eye and gently massage the eyelid to return the lens to the central position. If the problem persists, consult the eye care professional.
- Never use tweezers, sharp objects or fingernails to move, insert or remove lenses.

TAKING CARE OF LENSES

- DISINFECT** lenses each time they are removed to destroy harmful germs and to ensure safe and comfortable contact lens wear. Use a chemical, not heat, disinfection system.
- A separate **CLEANING & RINSING** step may be recommended by the eye care professional to remove contaminants from the lens surface. In addition, lenses may also require periodic enzymatic cleaning to remove protein build-up.
- Remove one lens first (always the same lens first to avoid mix-ups) and place the lens into the correct chamber of the lens storage case. Repeat this procedure for the second lens.
- The eye care professional should recommend an appropriate lens care system. CIBA Vision recommends the following for use with SoftPerm lenses:
 - Hydrogen peroxide disinfecting system: In a Wink® - Disinfectant, Neutralizing Rinse, Daily Cleaner and Refreshing drops
 - Chemical cleaning & disinfecting solution: SOLO-care®
 - Protein removal: Unzyme® Enzymatic Cleaner
 - Lubricating & rewetting drops: CLERZ® or Focus® Lens Drops
- To store lenses, clean, disinfect, neutralize, rinse, and leave them in the closed/unopened lens storage case until ready to wear. If the lenses have been stored in the unopened lens storage case for more than 24 hours, repeat the cleaning, disinfecting, neutralizing and rinsing procedures using fresh solutions prior to wearing the lenses.
- To prevent contamination and avoid serious eye injury, always empty and rinse the lens storage case after each use with fresh lens storage solution or hot tap water and allow it to air dry. Never use soaps, detergents or other cleaning agents. The storage case should be replaced regularly.

Important notes:

- DO NOT USE SALIVA, TAP WATER OR PRODUCTS DESIGNED ONLY FOR HARD OR RIGID GAS PERMEABLE LENSES.**
- NEVER USE HEAT** with lens care solutions or to disinfect lenses.
- NEVER USE ANY SOLUTION CONTAINING CHLORHEXIDINE**, as this may seriously interfere with lens wetting and comfort on the eyes.
- DO NOT USE CLEANERS WITH ISOPROPYL ALCOHOL.**
- Refer to the package inserts of each individual lens care product for important safety information and complete directions.

CARE FOR A DEHYDRATED LENS

If the SoftPerm contact lens is off the eye and exposed to air for 30 minutes or longer, its soft skirt will become dry and brittle.

To Rewet Lenses:

- Handle the lens carefully.
- Place the lens in the lens storage case and **SOAK** in a recommended sterile saline solution until the lens skirt returns to a soft state - at least one-half hour. Clean and disinfect the rehydrated lens prior to reinsertion.
- If after soaking, the lens skirt does not become soft, **DO NOT USE THE LENS**, but contact the eye care professional.

CARE FOR A STICKING LENS

A lens must move freely on the eye for the continued health of the eye. If the lens sticks (stops moving) on the eye, apply 2-3 drops of a sterile saline solution recommended for use in eyes. Wait until the lens begins to move on the eye before removing it. If non-movement of the lens continues, immediately consult the eye care professional.

GENERAL EMERGENCIES

If chemicals of any kind (household products, gardening solutions, laboratory chemicals, etc.) are splashed into the eyes:

- FLUSH EYES IMMEDIATELY WITH FRESH SALINE SOLUTION OR TAP WATER.**
- REMOVE AND DISCARD THE LENS, AND IMMEDIATELY CONTACT THE EYE CARE PROFESSIONAL OR VISIT A HOSPITAL EMERGENCY ROOM WITHOUT DELAY.**

CIBA Vision Corporation
11460 Johns Creek Parkway
Duluth, Georgia USA 30097
www.cibavision.com

CIBA Vision Canada, Inc.
2150 Torquay Mews,
Mississauga, Ontario
L5N 2M6 Canada

Print Date/Part Number: 11-2008 / D7352B
Printed In Canada

The table below provides a listing of Health Canada Device Identifier Codes.

Product Name	Additional Product Description	Device Identifier
SoftPerm	n/a	53

Additional Information

- The hydrogen peroxide disinfecting system In a Wink® has been discontinued.
- CIBA Vision now recommends Clear Care® for use with SoftPerm contact lenses.

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